

# Cardiosave Lithium-ion Battery Pack

## Determining Date of Manufacture

### Cardiosave Lithium-ion Battery Pack

A white barcode label is affixed to the top of each Cardiosave Lithium-ion Battery Pack. This barcode label provides information such as part number, serial number, and year and week of manufacture. Refer to the example below for barcode label placement.




Barcode Label Placement

Example barcode label placement

### Determining Date of Manufacture

Each barcode label provides information in the form of two barcodes. The first barcode is the Datascope Corp. part number of the Cardiosave Lithium-ion Battery Pack. The second barcode is coded with the year of manufacture, battery pack serial number, and week of manufacture.

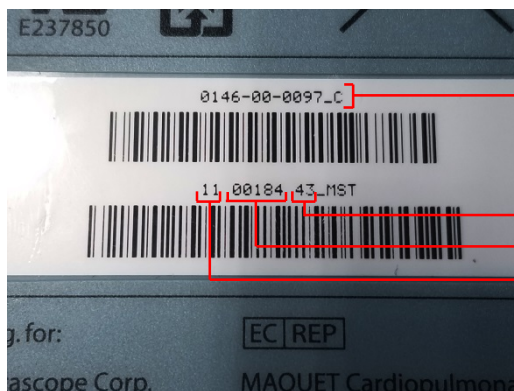

Datascope Corp. Part Number, **0146-00-0097**

Two Digit Week of Manufacture, **43**

Five Digit Serial Number, **00184**

Two Digit Year of Manufacture, **11**

In the above example, the Datascope Corp. part number for this battery pack is **0146-00-0097**. The serial number is **00184**, and it was manufactured in the week **43** of **2011**. The letters after the serial number vary.

#### NOTE

The year and week of manufacture will always be a two digit number, and the serial number will always be a five digit number.