IMPORTANT INFORMATION ON HOW TO REPLACE THE ELECTRODES

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 PLEASE READ THE FOLLOWING STEPS AND MAKE SURE YOU UNDERSTAND BEFORE TAKING ANY ACTIONS

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-  A) Remove the bag containing the AED from its location and place it on a firm surface to avoid any accidents happening either to you as a user or to the AED itself.

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-  B) Open the bag containing the AED and place the AED in a space that you can easily move and perform easy tasks.

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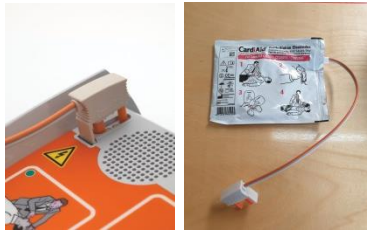
-  C) Open the AED itself with care and confirm that the electrodes are connected to the AED. The electrodes are placed in front of the first aid bag.

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-  D) With care, remove the electrode cable that connects to the AED from the plug and place the old electrodes somewhere safe, so you do not mix old electrodes with the ones replaced.

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-  E) Go ahead and after you remove the electrodes, replace them with the new units we have sent to you by connecting the cable of the NEW electrodes to the AED plug.

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-  F) Place the electrode pouch in front of the first aid bag as they were previously.

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-  G) Proceed with closing the cover of the AED, making a priority that cables do not over-extend and are **INSIDE** the unit. If not, to avoid having the cable cut or damaged, please reorganize cabling.

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-  H) Once that is done, proceed with placing the AED back in the bag of the AED and placed on its original location.

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 Dispose of the old electrodes. There is no need to send the old electrodes back to your local distributor.

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