Customer address

Gebrüder Martin GmbH & Co. KG
Product Management Implant Systems

23.04.2020 TH
Tobias Held

Tel.: +49 7461 - 706-428
Fax: +49 7461- 706-350
tobias.held@klsmartin.com

Safety notice - Additional product information about the screwdriver T6 Linos REF 26-975-39-07


Dear Sirs,

The purpose of this safety notice is to inform you about an additional product information initiated by Gebrüder Martin GmbH & Co.KG when using the T6 Linos screwdriver.

Description of circumstance

In the application, excessive mechanical torsional stress can cause the screwdriver to break at the working end.

Below you will find a corresponding illustration of such a breakage at the working end.

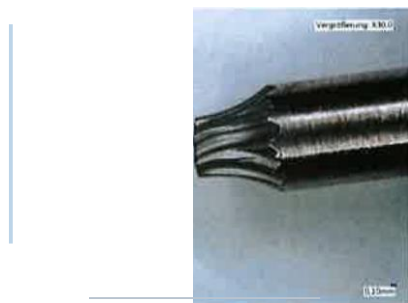Possible risks of breakage

The breakage and the fragment stuck in the screw head could lead to permanent damage of the tissue. It may not be possible to explant the plate.

Gebrüder Martin GmbH & Co. KG
Kl5 Martin Platz 1 · 78532 Tuttlingen/Germany
Postfach 60 · 78501 Tuttlingen/Germany
Tel. +49 7461 706-0 · Fax +49 7461 706-193
info@klsmartin.com · www.klsmartin.com

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KLsmart1n

GROUP

NOTE for correct application of the T6 Linos screwdriver REF 26-975-39-07

If you cannot use an alternative product to the one mentioned above, we recommend performing the insertion of the multidirectional smartDrive® locking screws only until you feel a noticeable locking effect. It is not necessary to continue the rotation movement after this resistance; in worst case, this could result in damage of the screwdriver. The rotary motion itself should only come from the hand, rotation from the upper arm should be strictly avoided.

In addition to the product listed here, the Safety Notice must be considered for the following additional articles:

Material	Description German	Description English
26-975-33-07	smartDrive Schraubendreher kurz T5 drehb	smartDrive Screwdriver T5 rotating short
26-975-36-07	smartDrive Schraubendreher kurz T6 drehb	smartDrive Screwdriver T6 rotating short
26-975-37-07	smartdrive Schraubendreher T6	smartDrive Screwdriver T6
26-975-38-07	smartDrive Schraubendreher kurz T5	smartDrive Screwdriver T5 short
26-975-39-07	smartDrive Schraubendreher kurz T6	smartDrive Screwdriver T6 short
26-975-40-07	Schraubendreherklinge smartDrive T6, L	Screwdriver blade smartDrive T6, L

Transmission of safety notice "Additional product information"

Please ensure that all users of the product mentioned and other affected people in your organisation who need to be informed are aware of this safety notice for additional product information. If you have submitted the products to third parties, please forward a copy of this safety notice "Additional product information" or inform the contact persons below. Please keep this information at least until you have informed all the people in your company who has to be informed.

Gebrüder Martin GmbH & Co.KG

Your responsible health authority for medical devices has received a copy of this safety notice "Additional product information about the screwdriver T6 Linos REF 26-975-39-07".

Gebrüder Martin GmbH & Co. KG
Kl5 Martin Platz 1 · 78532 Tuttlingen/Germany
Postfach 60 · 78501 Tuttlingen/ Germany
Tel. +49 7461706-0 · Fax +49 7461706-193
info@klsmartin.com · www.klsmartin.com

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KLSmart1n GROUP

Please do not hesitate to contact us in case of any further questions.

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KLsmart1n

GROUP

Safety notice - Additional product information about the screwdriver T6 Linos REF 26-975-39-07

Annex

Reply-Fax to +49 7461 706-350
to Gebrüder Martin GmbH & Co.KG

Customer _____

Customer-No. _____

NOTE for correct application of the T6 Linos screwdriver REF 26-975-39-07

If you cannot use an alternative product to the one mentioned above, we recommend performing the insertion of the multidirectional smartDrive® locking screws only until you feel a noticeable locking effect. It is not necessary to continue the rotation movement after this resistance; in worst case, this could result in damage of the instrument. The rotary motion itself should only come from the hand, rotation from the upper arm should be strictly avoided.

In addition to the product listed here, the Safety Notice must be considered for the following additional articles:

Material	Description German	Description English
26-975-33-07	smartDrive Schraubendreher kurz T5 drehb	smartDrive Screwdriver T5 rotating short
26-975-36-07	smartDrive Schraubendreher kurz T6 drehb	smartDrive Screwdriver T6 rotating short
26-975-37-07	smartdrive Schraubendreher T6	smartDrive Screwdriver T6
26-975-38-07	smartDrive Schraubendreher kurz T5	smartDrive Screwdriver T5 short
26-975-39-07	smartDrive Schraubendreher kurz T6	smartDrive Screwdriver T6 short
26-975-40-07	Schraubendreherklänge smartDrive T6, L	Screwdriver blade smartDrive T6, L

We herewith confirm:

Receipt of the letter safety note "Additional product information about the screwdriver T6 Linos REF 26-975-39-07" from Gebrüder Martin GmbH & Co.KG from April 2020.

Place, Date

Name/ Position of signatory

Organisation/Stamp/Signature of signatory

Gebrüder Martin GmbH & Co. KG
KLS Martin Platz 1 · 78532 Tuttlingen/Germany
Postfach 60 · 78501 Tuttlingen/Germany
Tel. +49 7461706-0 · Fax +49 7461706-193
info@klsmartin.com · www.klsmartin.com

Registergericht: Stuttgart
Handelsregister: HRA 450196
Ust.-IdNr.: DE 1429 30777

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Commerzbank AG, Filiale Tuttlingen: BLZ 694 400 70 · Kto. 0241215300