



**Urgent Field Safety Notice (FSN) for FD-8000: Premature wear of brake-arm levers**

**FD-8000 units sold between 01/01/2015 and 31/12/2017 require an urgent repair to replace the brass levers with steel levers. This instruction explains how to do the repair.**

1. Open arm brake cover to expose the levers.

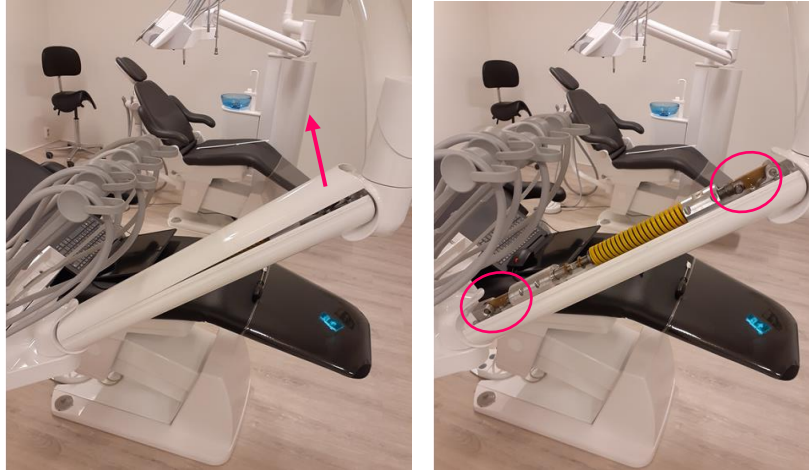

2. Position the bridge. Raise the bridge on the patient chair headrest. Raise the bridge until the axles are loose and removable.

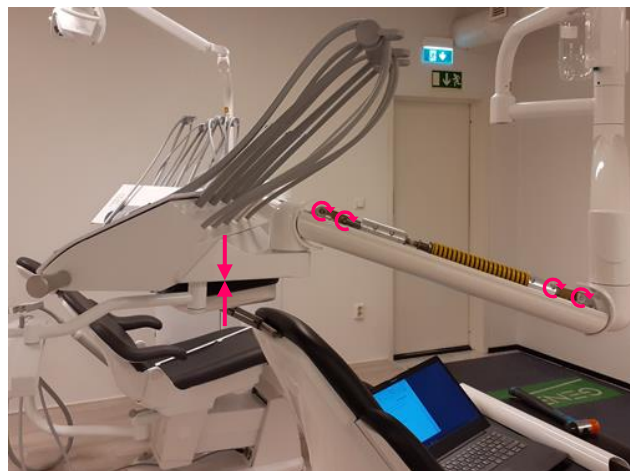

3. Remove the back axles. Save the axles and lock rings.




**Urgent Field Safety Notice (FSN) for FD-8000: Premature wear of brake-arm levers**

- 4. Remove the back brass lever. Clean away the dust and old grease. Apply clean grease.



- 5. Put back the axles with the replacement steel lever.

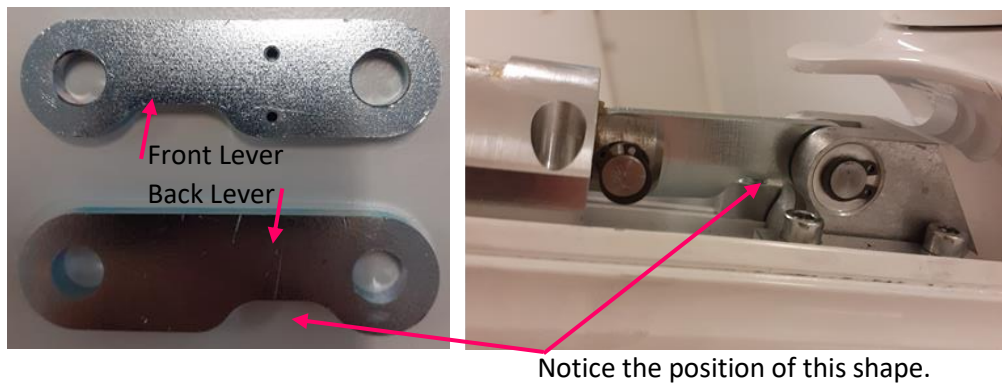

- 6. Repeat steps 3-5 for the front lever.




**Urgent Field Safety Notice (FSN) for FD-8000: Premature wear of brake-arm levers**

7. Put on the unit power and air and test the brake arm motion.



8. Put back the cover.

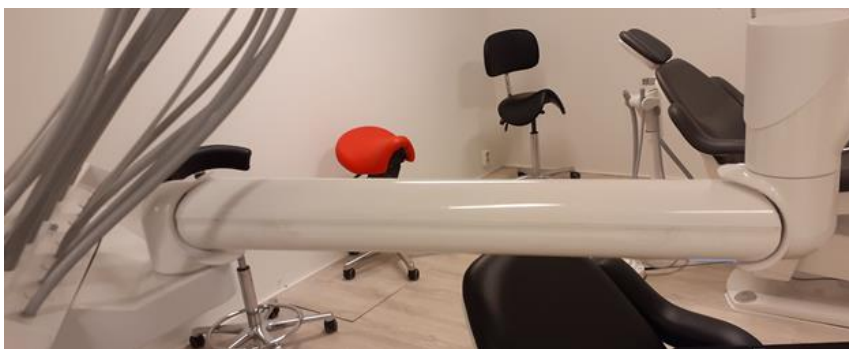

9. Record the repair information and return to Finndent (.....@Finndent.com). The list of serial numbers is attached to the Field Safety Notice.

Repair Date	Serial Number(s)	Clinic	Technician

**The brake-arm levers are repaired.**