Urgent Field Safety Corrective Action Notice

Date: 13th October 2023

Field Safety Notice Number: RFA-2

Dear Customer,

Our records indicate that we have supplied your company with **The Insides Driver**, including a specific manufacturing batch of units that **could present a risk to users**. The Insides Driver is a component of The Insides System and The Insides Patient Education Model.

The Insides Company received notice on 12th October 2023 from a customer that a single unit of The Insides Driver overheated, caught fire, and exploded while being recharged from a low battery state.

Our initial evaluation of worst-case scenarios has concluded that there could be a risk of injury to users of a specific manufacturing batch of The Insides Driver during the recharging operation. Our evaluation suggests these devices do not pose a risk if not in use and not being recharged. Please remember that The Insides Driver is labelled for single patient use only.

This letter is official notice that The Insides Company is requesting action from you to prevent a possible hazard to persons and equipment.

Our requested actions for the affected units of The Insides Driver listed on Page 3 are:

1. Please communicate this Field Safety Corrective Action Notice to all relevant staff and users.
2. Please do not distribute or recharge affected units that are in your possession.
3. Where applicable, please communicate this Field Safety Corrective Action Notice to third parties you have supplied affected units to.
4. Please return all affected units to the return address specified on the Customer Response Form.
5. Please complete the **Customer Response Form**, on Page 4, and email it to tina.mason@theinsides.co
6. Upon receipt of the completed Customer Response Form, **The Insides Company will arrange replacements for the affected devices.**

We sincerely regret any inconvenience that this Urgent Field Safety Corrective Action Notice may cause, however, we greatly appreciate your understanding as we take actions to ensure patient safety and continued customer satisfaction.

The undersigned confirms that the Urgent Field Safety Corrective Action Notice will be notified to the appropriate Regulatory Agency. Should you have questions or need additional information, please contact The Insides Company.

Kindest Regards

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Urgent Field Safety Corrective Action Notice

Date: 13th October 2023

Field Safety Notice Number: RFA-2

Date:	13 October 2023
Manufacturer Single Registration Number (SRN)	NZ-MF-000011306
Product Affected	<p>The Insides Driver Part Number: PS001</p>  <p>The Insides Driver is a component of:</p> <p>The Insides System Part Numbers: PS005 = UK/EU and PS006 = NZ/AU/ZA)</p>  <p>The Insides System Patient Education Model (PEM) Part Number: MK-1R001</p> 
Product Issue:	The Insides Driver overheated while being recharged. This resulted in the device exploding, scattering parts to a 1 meter radius, with some parts on fire. Fumes were also emitted.
Resolution:	While the investigation into the root cause of this issue is being conducted, we request that you discontinue use of all affected units, listed below, with immediate effect.

Urgent Field Safety Corrective Action Notice

Date: 13th October 2023

Field Safety Notice Number: RFA-2

Affected LOTS / Serial Numbers:	PS005 The Insides System			
	The insides System UDI	The insides System Serial Number	The insides Driver UDI	The insides Driver Serial Number
	01094219054470341725040310000023121050000FF	050000FF	0109421905447010172504031000002272101000274	01000274

Customer Response Form

Reference: Field Safety Corrective Action Notice: RFA-2

Our records indicate that the The Insides Driver (included in PS005/PS006 The Insides System and Patient Education Model) shown below was provided to you. Please verify that you have the listed device and complete the information below.

PRODUCT	SN/LOT	CURRENT LOCATION
PS005	050000FF	
TOTAL AFFECTED PRODUCT 1		

Please check the appropriate boxes below:

- I acknowledge receipt of the Field Safety Corrective Action Notice dated 13 October 2023 related to the above product. We have read and we understand the communication and the required actions.
If checked : please provide information below.

Field Safety Notice Receipt and Customer Response Form Completion

Organisation	GD MEDICAL PHARMA BV		
Contact Name / Title			
Address (no PO boxes, please)			
City, State, Zip			
Phone Number		Fax:	
E-Mail Address:			
Signature		Date	

- We have supplied potentially affected product to another organisation.
If checked : please provide new organisation information below.

<u>New</u> Organisation Name			
Contact Name / Title			
Address*			
City, State, Zip			
Phone Number		Fax:	
E-Mail Address:			

PLEASE RETURN YOUR COMPLETED FORM BY EMAIL TO: tina.mason@theinsides.co

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