	<b>Formulaire</b>	FO-QUA-030-V002
	<b>SAFETY INFORMATION</b>	N° page 1/1

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Dear customer

A safety corrective action measure has been initiated for the products below:

Designation	References
F1 2000 Chair	11500ST, 11500STTE, 11500TE, 11500DE, 11500DTE
F1 Ergo 2000 Chair	11550, 11550TE, 11550DTE
F1 Neo Chair	11700, 11700DE, 11711, 11711DE
F1 Neo Ergo Chair	11750, 11750DE, 11761, 11761DE
F1 3000 Chair	11004TE, 11004DTE
F1 3000 Ergo Chair	11007TE, 11007DTE

#### Description of the problem giving rise to the safety information and potential risk

In conjunction with the company FIMET OY, manufacturer of the F1 armchair base, it was detected that a weld in the structure of the seat frame could present a possible failure and cause the backrest to tilt backwards with a risk of fall of the patient.

#### Action to do mandatory by the distributor

We ask you to distribute this safety information without delay to all customers who are equipped with a F1 chair, to take the necessary measures and to the customers to complete the questionnaire.

#### User action

The probability of occurrence is very low but we must nevertheless implement all precautionary measures. While waiting for this preventive security solution, we advise you to restrict the use of Trendelenburg as indicated below:

To enable us to organize the shipment of a safety kit, we must ensure that you still have this type of chair and ask you to kindly acknowledge receipt of this safety information upon receipt by responding to the questionnaire below:

**Reception confirmation**  
**Questionnaire**

Please note that this materiovigilance declaration was made to the competent authorities.

Our quality department is available at the dedicated email address: [assistance@capronpodologie.com](mailto:assistance@capronpodologie.com)

Our aim is to provide you with a professional service, best regards.